



CONSCIENCE™

## Conscience Brand Conscientious Masks



EUA202354

**Intended Use:**

Intended to be worn by healthcare personnel in healthcare settings to protect both the patient and the operating personnel from transfer of microorganisms, body fluids and particulate materials during COVID-19 virus pandemic.

### Basic Information:

|   |   |
|---|---|
| <b>Product Name</b>                                 | Particle Filtering Half Mask  |
| <b>Model</b>  | JW-017  |
| <b>Type of Respirator</b>                           | Headbands type, passed fit tests.   |
| <b>Execution Standard</b>                           | EN 149:2001+A1:2009   |
| <b>Classification</b>                               | FFP2 NR   |
| <b>Particulate Filtration Efficiency (PFE)</b>      | ≥95% @ 0.3 micron particulate of both sodium chloride (NaCl) and dioctylphthalate (DOP)   |
| <b>Bacterial Filtration Efficiency (BFE)</b>        | ≥98% @ 3.0 micron particulate of sodium chloride (NaCl)   |
| <b>Breathing Resistance</b>                         | ≤0.7 mbar (70 mmH <sub>2</sub> O) @ 30 L./Min. of inhalation<br>≤2.4 mbar (240 mmH <sub>2</sub> O) @ 95 L./Min. of inhalation<br>≤3.0 mbar (300 mmH <sub>2</sub> O) @ 160 L./Min. of exhalation |
| <b>Total Inward Leakage</b>                         | ≤11%  |
| <b>Compatibility with Skin</b>                      | No irritation or any other adverse effect to health   |
| <b>Carbon Dioxide Content of The Inhalation Air</b> | Not exceed an average of 1.0% (by volume)   |

## Attention:

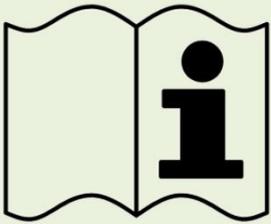
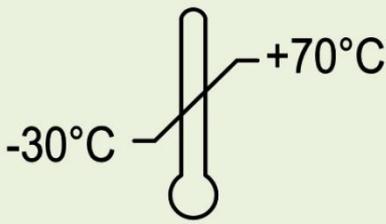
1. This is a respiratory protective device designed to achieve a very close facial fit and very efficient filtration of non-oily & oily airborne particles. The edges of the respirator are designed to form a seal around the nose and mouth. When subjected to careful testing, the respirator blocks at least 95% of very small (0.3 micron) test particles.
2. People with chronic respiratory, cardiac, or other medical conditions that make breathing difficult should check with their health care provider before using this respirator because this respirator can make it more difficult for the wearer to breathe.
3. If your respirator is damaged or soiled, or if breathing becomes difficult, you should remove the respirator, discard it properly, and replace it with a new one. To safely discard your respirator, place it in a plastic bag and put it in the trash. Wash your hands after handling the used respirator.
4. It's not designed for children or people with facial hair. Because a proper fit cannot be achieved on children and people with facial hair, the respirator may not provide full protection.
5. Store indoor. Relative humidity less than 80%; good ventilation; avoid corrosive gas and high temperature.
6. 2 years guarantee under the specified storage conditions and methods.

## HOW TO USE:



1. Facing the no-nose-clip side of the respirator with the nose clip on the top, hold the respirator by hand to fix it on the face to cover the mouth and the nose;
2. Pull the two headbands over the top of the head, place the bottom headband under the ears behind the neck, and place the top headband over the ears behind the neck;
3. Place the fingers of both hands in the middle of the metal nose clip, press inwards, and move the fingertips along the nose clip to the sides until the nose clip is pressed into the shape of the bridge of the nose;
4. Take several breaths to make sure that there is no air leakage between the respirator and the face. If air leaks from the edge of the respirator, readjust the headband; if air leaks from the bridge of the nose, readjust the nose clip according to step 3.

## STORAGE CONDITIONS:

|   |   |  |   |
|---|---|--|---|
|  |  |  |  |
| Instruction for Use is in the box   | The storage temperature is during -30°C ~ +70°C                                   | The relative storage humidity is <80%  | The shelflife is 2 years  |

Manufacturer: **Jiangmen Nostop Electric Co., Ltd.**

Add.: 1<sup>st</sup> & 5<sup>th</sup> Bldg., No. 15, Jinhui Road, Jianghai

District, Jiangmen City, Guangdong Province, China

Tel: +86.750. 2635680, 2635681, 3761838

Website: [www.nostop.cn](http://www.nostop.cn), E-mail: [stony@nostop.cn](mailto:stony@nostop.cn)